

Breath Alcohol Screen

The **Breath Alcohol Screen** offers users a simple, convenient method to detect alcohol use and to estimate levels of alcohol intoxication. The device uses chemical alcohol detection technology with 50 years of proven success in law enforcement and corrections.



To perform a test with the Breath Alcohol Screen, the donor squeezes the tube to crush the ampoule and then blows into the end of the tube for 12 seconds.

If alcohol is not present at the test's threshold, the yellow crystals remain unchanged. [1]

Alcohol in the breath that meets or exceeds the test's threshold turns the yellow crystals into a pale blue/green colour within two minutes of the donor providing a breath sample. [2]

Hampton Knight can provide competency training, testing protocols and consent forms to ensure your testing process is robust and legally defensible.



[1]



[2]



Key Features

- ✓ Compact, plastic tube construction allows test to be carried anywhere
- ✓ Pass/fail design detects alcohol at different threshold levels
- ✓ Available in three threshold levels .02, .05, .08

For further information or to order this product please call **01827 65999** quoting product code: **BAS001**